

**Feline Herpesvirus Antigen Lateral Flow Assay Kit**

**Catalog No:** E-AD-C102

40T

This manual must be read attentively and completely before using this product.

If you have any problems, please contact our Technical Service Center for help.

Toll-free: 1-888-852-8623 Tel: 1-832-243-6086 Fax: 1-832-243-6017

Email: [techsupport@elabscience.com](mailto:techsupport@elabscience.com)

Website: [www.vetassay-elab.com](http://www.vetassay-elab.com)

Please kindly provide us the lot number (on the outside of the box) of the kit for more efficient service.

## Test principle

This kit uses the principle of Immunochromatography assay for the qualitative detection of Herpesvirus antigen in secretions of (mouth, eyes, nose and anus) of Feline. The sample will move together with the colloidal gold marker along the chromatography membrane. If Feline Herpesvirus (FHV) antigen exist in the samples, it will combine with the colloidal gold marker to make the detection line appear a color. Otherwise, it will not show the color.

## Kit components

Item	Specification
Detection Card (with disposable dropper )	40T
Sample Diluent	40 vials
Cotton Swab	1 package
Manual	1 copy

Note: All reagent bottle caps must be tightened to prevent evaporation and microbial pollution.

## Notes

1. FOR RESEARCH USE ONLY. Please read the manual carefully before use, changes of operation may result in unreliable results.
2. Do not use product out of date or in a broken aluminum foil, it is disposable and cannot be used repeatedly.
3. The detection card should be brought to room temperature before opening after take it out from the refrigerator. The opening detection card should be used as soon as possible.
4. Please do not use but not limited to the following liquids for negative control: water, PBS.
5. The tested sample should be fresh and clear. Avoid of using samples of turbidity, polluted, high hemolysis or abnormal viscous.
6. Avoid of touching the chromatography membrane of the sample well and test well.
7. The waste of experiment should be considered as contaminant, and must be properly handled according to the local regulations.
8. **Each reagent is optimized for use in the E-AD-C102. Do not substitute reagents from any other manufacturer into the test kit. Do not combine reagents from other E-AD-C102 with different lot numbers.**

## Storage and expiry date

**Storage:** Store at 2-30°C. With cool and dry environment, avoid freeze.

**Expiry date:** expiration date is on the packing box.

## Sample preparation

1. **Secretions of mouth, eyes, nose and anus:** Take the secretion with a cotton swab to make the cotton swab completely wet. Immediately, insert the cotton swab into the **Sample Diluent**, stir the swab until the sample is dissolved into the sample diluent fully. Discard the cotton swab after wiping it against the wall of the tube. Make sample solution stand for later use.


## Assay procedure

Allow all kit components and sample to reach room temperature (25 °C) prior to testing.

1. Tear the aluminum foil bag of the detection card and take out the detection card, and put it on a smooth, clean table.
2. Take the clear sample supernatant with the disposable dropper, add 3 drops (about 60 µL) of sample solution to the sample well (S) vertically and slowly (Avoid foaming).
3. Incubate 10 minutes and then judge the results immediately.

## Judgment of result

1. **Negative:** Only the control line region (C) shows a line in the observation well.
1. **Positive:** Both the test line region (T) and the control line region (C) show a line in the observation well.
2. **Invalid:** No line shows in the observation well of the control line region (C).

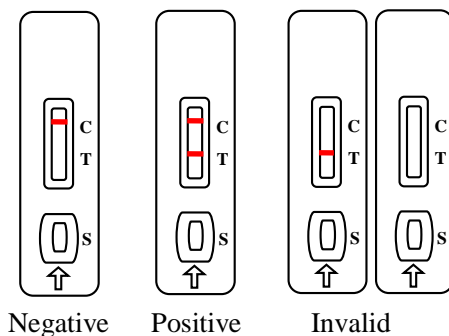

## Limitations

1. This kit can only be used for qualitative detection of FHV antigen of Feline.
2. The detection results of this kit are only for reference. For confirmation of the result, please combine the symptoms and other methods of detection, this detection cannot be used as the only criteria for result.